cálido confort

PM-145

Tensiómetro de brazo

Arm pressure monitor

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DIRECTIONS FOR USE

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INSTRUCTIONS

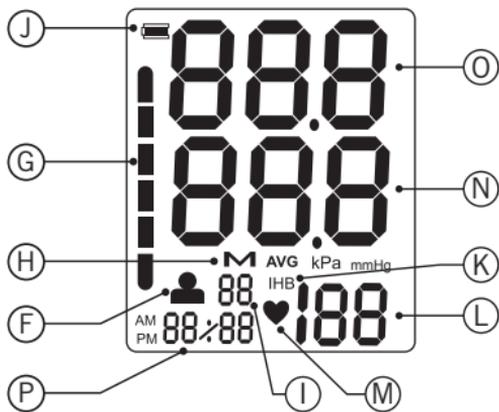
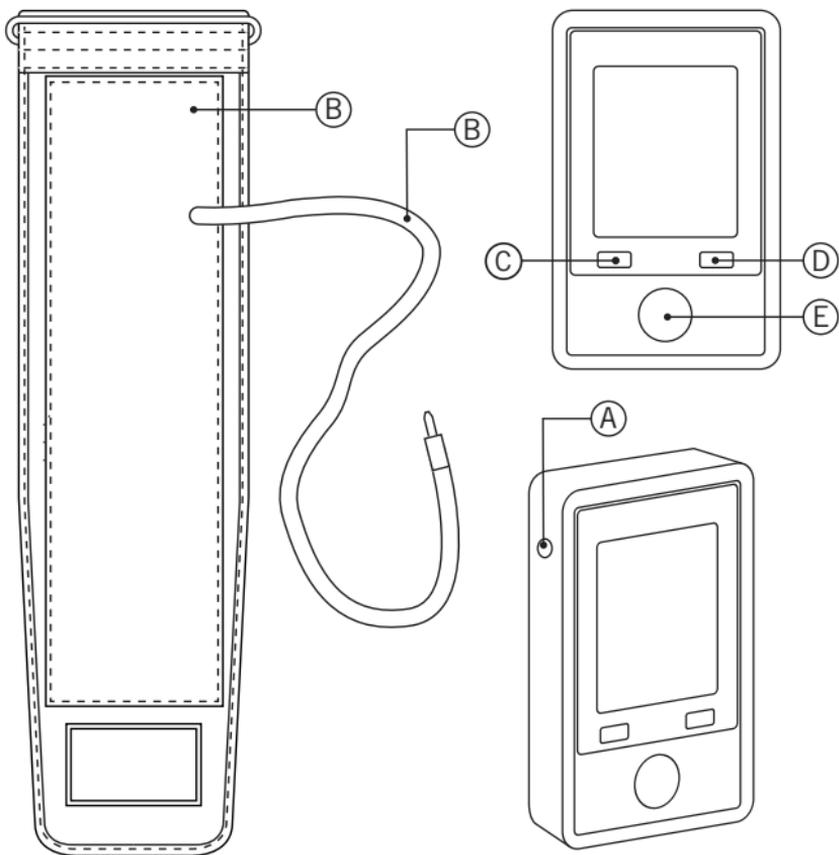
Fig. 1

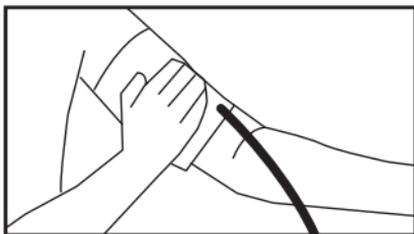
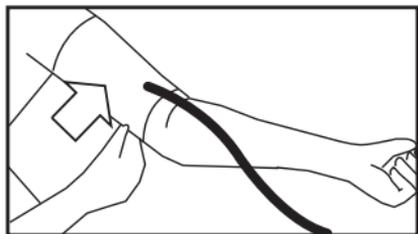
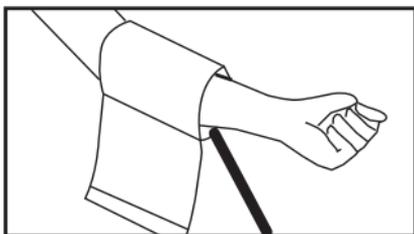
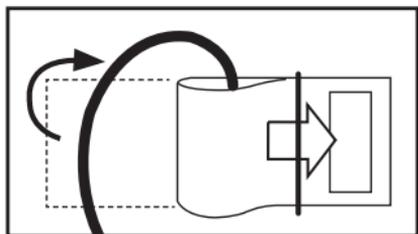
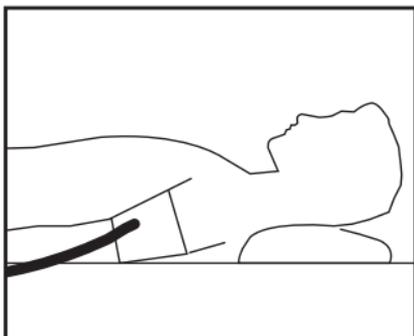
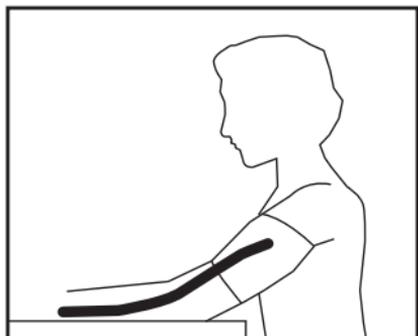
Fig. 2

AVISO IMPORTANTE

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WARNING

Please read the instructions for use carefully prior to using the product. Store these in a safe place for future reference.

The arm blood pressure monitor is used to read and monitor your blood pressure values in a non-invasive manner.

Children should always use this product under supervision, this product is not a toy.

Use caution when handling the cuff cable to prevent asphyxiation.

Do not use the blood pressure monitor on newborns, pregnant women or patients with preeclampsia.

This is solely intended for domestic use.

The readings obtained are for information purposes only. You should inform your doctor of these readings. Never take any medical decision without consulting with your doctor (such as modifying your medication).

Do not use the appliance close to strong magnetic fields.

Do not squeeze the cuff tube when taking a reading.

The blood pressure monitor should not be used alongside any high-frequency surgical equipment.

In order to prevent any risk of danger, do not handle the product. Only qualified technical personnel from the brand's Official Technical support service may carry out repairs or procedures on the device.

Any incorrect use or improper handling of the product shall render the warranty null and void. Always take your device to an authorised technical support service for repairs.

ENGLISH

The product packaging contains:

- Main device
- Adjustable cuff with air tube
- Instruction manual

Parts and components:

- A. Air connector
- B. Air tube and adjustable cuff
- C. Settings button
- D. Memory button
- E. On/off button

LCD screen:

- F. User
- G. OMS indicator
- H. Memory symbol
- I. Memory number
- J. Low battery indicator
- K. Irregular pressure/ arrhythmia indicator
- L. Pulse
- M. Pulse symbol
- N. Diastolic blood pressure
- O. Systolic blood pressure
- P. Year/month/day, time

* Batteries not included.

INSTRUCTIONS FOR USE

Place 3 AA-size batteries in the appliance. Remove battery compartment cover. Ensure that they are placed in accordance with the polarity indicated on the appliance. Close the battery compartment. Prior to taking any reading, check that the low battery (L) indicator is not displayed. Likewise, check the device for defects before each use.

ENGLISH

First time using the device. User, date and time programming

20XX will start to flash on the screen when you turn on the device for the first time or change the batteries. You can increase the latter digits by pressing the memory button (D), selecting any year from 2011 to 2030. Press the SET button (C) to confirm the year selected.

The data corresponding to the month and date will then appear on the screen. Start off by selecting the month, pressing the memory button to move between 1 and 12 and confirm with the settings button. Do the same for the date, selecting them by pressing the memory button and confirming with the settings button.

Once you have confirmed the date, the device will then move to the time settings, with the time flashing. Press the memory button to increase the time, selecting any time from 0 to 23, confirming with the settings button. Once you have confirmed the time, the digits corresponding to the minutes will start to flash, which you can select by pressing the memory button, selecting from between 00 and 59, and confirming the time selected with the settings button.

Taking a reading.

Fit the cuff onto the top of your left arm and adjust as shown in Fig. 1. Rest for 5 minutes before starting the reading. Otherwise, there may be inaccuracies.

You can take the pressure lying down on your back or sitting with your arm on a table at the same height as your heart, as shown in Fig. 2. Rest your hand on the table, palm upwards.

We recommend taking the reading from the left arm. However, if this cannot be done, you can do it with your right arm, remembering that all readings must be taken from the same arm.

Adjust the cuff on your arm so that the air tube runs towards your hand. Make sure to leave 2-3 cm between the cuff and your inner elbow and that you can slide your fingers easily between your arm and the cuff.

The arm sizes with which the product can be used are given on the cuff. If the cuff does not fit around your arm, do not use this device.

Any piece of clothing that may be exercising pressure on your arm must be removed.

ENGLISH

Steps for taking a reading:

- 1) Press the on/off button (F).
- 2) The screen will display all the symbols of the same. You will then feel a slight pressure on the cuff and the reading will commence. Do not move or speak.
- 3) Once the pressure required is reached, it will start to deflate. If this is not sufficient to take a reading, the device will automatically reinflate to a greater pressure.
- 4) When the device detects a signal, the heart symbol will start to flash on the screen.
- 5) Once the reading has been completed, the results of the reading will appear on the screen.
- 6) The reading will be displayed on the screen until the device is turned off. If you do not press any button for 3 minutes, the device will automatically switch off in order to save energy

PLEASE NOTE: The irregular pressure indicator symbol (K) will appear along with the results of the reading if any arrhythmia has been detected in the reading.

Rapid deflation during reading

If you feel unwell during the reading or you wish to stop the device for any reason, you can press the on/off button for the device to quickly release the air from the cuff and the device will return to standby mode.

Record of readings and viewing of readings on the device memory

The device automatically saves 2x90 reading results in its memory. Once this goes over 60 readings, the oldest reading is replaced with a newer one.

Pressing the memory button (D) with the device turned off displays the average of the last 3 readings on the screen. Pressing the memory button again displays the last reading taken, as well as the subsequent readings every time that you press the memory button again.

Delete reading record from the memory

If you are sure that you want to delete all the records saved in the device, press the MEM memory button for 4-6 seconds while the device is in the read mode. The screen will display “dEL no”, press the memory

ENGLISH

button again and the screen will display “dEL yES”. If you are sure that you want to delete the information saved in the memory, press the On/Off button to delete.

Select user

Press the SET button (C) in order to choose between user 1 and 2, with the product turned off. The screen will display the user selected and the memories recorded on the device for this user. Press the button again to move from one user to another, depending on who you want to check the memories or take a reading for.

Delete reading record from the memory

If you are sure that you wish to delete all of the memories saved in the device, press the settings button 7 times with the device turned off, until CL is displayed on the screen. Then press the on/off button, and the letters CL will flash 3 times and the reading records will be deleted. Then press the memory button and NO will appear on the screen, meaning that there are no memories saved on the device.

To ensure that the values are comparable, always take a pressure reading at the same time of day.

If you want to perform more than one reading for the same person, wait 5 minutes between readings.

Repeat the reading if you do not trust the validity of the reading values. In the event that you believe that the results are not correct, take the appliance to a Technical Support Service.

As this is a home use product, it does not need to be calibrated.

ENGLISH

PROBLEM		Solution
When pressing the on/off button, nothing appears on the screen.	<ul style="list-style-type: none"> • The batteries have expired. • The polarity with which the batteries were placed is not correct. • The batteries are not making proper contact. 	<ul style="list-style-type: none"> • Change the batteries with new ones. • Insert the batteries properly. • Wipe the cavity where the batteries sit with a dry cloth.
The pressure exerted by the cuff is not constant.	Have you remained stationary during this process?	Stay still and silent during the reading.
The reading is extremely high or low.	<ul style="list-style-type: none"> • Is the cuff at the same height as the heart? • Is the cuff placed properly? • Have you remained stationary during this process? 	<ul style="list-style-type: none"> • Make sure your posture is correct during the reading. • Position the cuff properly (Fig. 1) • Stay still and silent during the reading.
The pulse reading is too high or too low.	<ul style="list-style-type: none"> • Have you remained stationary during this process? • Have you taken the reading just after doing physical exercise? 	<ul style="list-style-type: none"> • Keep still and silent during the reading. • Take the reading again after 5 minutes of rest.
The batteries expire quickly.	The batteries are faulty.	Use alkaline batteries from well-known manufacturers.
The following appears on-screen: Err	<ul style="list-style-type: none"> • Weak signal or pressure has suddenly changed. • An error has taken place during inflation. 	<ul style="list-style-type: none"> • Ensure that the cuff is well positioned and the air tube properly fitted. Repeat the reading. • Stay still and silent during the reading. • Adjust the cuff so that it fits better.
The following appears on-screen: Lo	Low battery.	Replace the old batteries with new ones.

ENGLISH

Please contact an authorised technical service centre or contact us directly via the website www.daga.eu if you have any query related to use, maintenance or configuration of the device.

PRODUCT CONSERVATION

Remove the batteries when you do not intend to use the product for an extended period of time to prolong its shelf-life.

The product must be stored in a dry location in its original packaging. Protect the device against bangs, moisture, sudden changes of temperature and solar radiation.

Clean the product, including the cuff, with a damp cloth and dry afterwards. Clean every 3-6 months. If the product is to be used by more than one person, it should be cleaned after every use.

Do not immerse the product in water or other liquid.

Check the different parts of the product regularly and take it to an authorised Technical Support Service in the event of any damage. Never use spares from other brands.

TECHNICAL DATA

Measurement method	Oscillometric. This means that the product detects the movement of the blood through the brachial artery, converting blood pressure into a digital readout.
Measuring Range	0-299 mmHg 40-180 beats per minute
Precision of the indication	±3 mmHg Pressure Pulse ± 5% of indicated value
Memories	2x90
Operating Conditions	From +10 °C to 40 °C Moisture greater than 15% and less than 90%
Transport and storage conditions	From -20°C to 60°C Moisture greater than 10% and less than 95%
Power	3 X 1.5V batteries (not included)

ENGLISH

Please contact an authorised technical service centre for any abnormal operation incidents.

This product complies with the Medical Devices Directive 93/42/EEC, including the European standards EN1060-1 (Non-invasive sphygmomanometers, part 1: general requirements) and EN1060-3 (Non-invasive sphygmomanometers, part 3: Additional requisites applicable to blood pressure measurement electro-mechanical systems).

PRODUCT DISPOSAL

Do not dispose of the product at the end of its useful life. Contact your nearest collection centre in accordance with the waste electrical and electronic equipment (WEEE) Directive 2012/19/EU to contribute to the protection of the environment.

Request detailed information regarding their current methods of disposing of packaging and scrappage of used devices from your distributor.

We hope that you will be satisfied with this product.

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Elimination of waste according to European Directive 2002/96/EC on waste electrical and electronic equipment

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The product may contain batteries. Remove them before disposing of the product and dispose in special containers approved for this purpose.

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Shenzhen Kingfield Technology Co., Ltd.

Section C1-2, FuHai Industrial Zone, Fuhai Road FuYong Town, Bao'an District, 518103 Shenzhen City, GuangDong, China
www.kingfield.com

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Cachet de l'établissement vendeur

ENGLISH / WARRANTY REPORT

B&B TRENDS, SL. guarantees compliance of this product for the purpose for which it is intended for a period of two years the entire Spanish territory. In the case of breakdown during the term of this warranty, you are entitled to repair or to the replacement of the product at no charge if the former is unavailable, unless one of these options prove impossible or disproportionate. In this case, you can then choose a reduction in price or cancellation of the sale, which must be dealt with directly with the sales vendor. This also covers replacement of spare parts provided that the product has been used according to the recommendations specified in this manual for both cases, and has not been tampered with by any third party that is unaccredited by **B&B TRENDS, SL.** This warranty does not affect your rights as a consumer in accordance with the provisions of Law 23/2003 dated 10 July 2003 on warranties on the sale of consumer goods in the Spanish territory.

USE OF WARRANTY

Consumers should contact a technical service centre authorized by **B&B TRENDS, SL.** for repair of the product as any tampering thereof by persons not authorized by Daga, as well as misuse or inappropriate use, will lead to the cancellation of this warranty. The warranty must be fully completed and delivered along with the receipt or delivery docket for the effective exercise of rights under this warranty. Claims **must take place within 2 years** from deliver of the project according to the provisions of the warranty, and the consumer must inform the sales vendor of the lack of conformity, provided that such claim is made within the statutory period of warranty. This warranty, as well as the invoice, receipt, or the delivery order must be retained by the user to facilitate the exercise of these rights. For technical service and after-sales care outside the Spanish territory, please submit your query to the point of sale where you purchased the item or by contacting **B&B TRENDS, SL.**

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JULEN S.A.T.